

Crowd-sourced Naloxone with NaloxoFind

Training



Note to instructor: The NaloxoFind Trainer app operates on a dedicated training platform and can be used for demonstration purposes. Available on Apple and Google Play.

Background

- NaloxoFind is a free app for Apple and Android smartphones.
- The app enables anyone around an overdose victim to send anonymous requests to nearby naloxone carriers.
- When a request is made, all registered naloxone carriers nearby are notified and the requestor is prompted to dial 911.
- If a registered naloxone carrier accepts an alert request, the app automatically guides them to the patient, enabling timely administration naloxone.
- NaloxoFind also shows registered public naloxone locations (e.g., wall mounted naloxone) within a two mile radius.
- Purpose of NaloxoFind is to augment not replace the 911 EMS system.



Installation

NaloxoFind is available on the Apple App Store and on Google Play – search for "NaloxoFind" or point your smartphone camera at the barcode below.



Point your smartphone's camera at the above picture to install NaloxoFind.




NaloxoFind User Types

- Anonymous bystanders are users who have downloaded the app but have not registered as naloxone carriers.
- Registered naloxone carriers have registered with the app and will be alerted if a nearby user requests help.



Bystander Usage

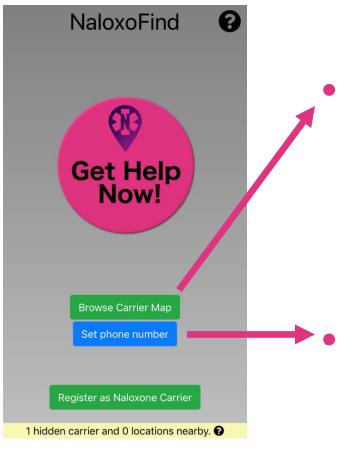


Browse registered locations; registered carriers are hidden until a request is accepted.

Set phone number that will be displayed to responders when Get Help button is pressed.




Bystander Usage – Get Help

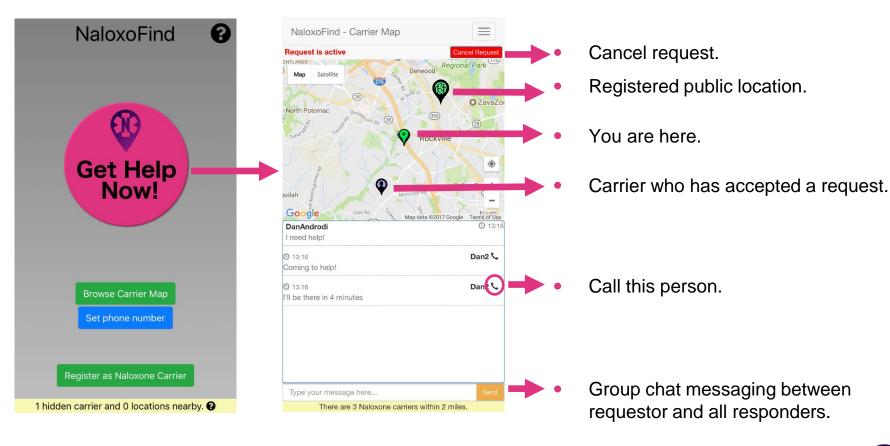




Naloxone Carrier - Registration

Verify that you are a naloxone carrier by scanning the barcode on your naloxone using your camera's phone.

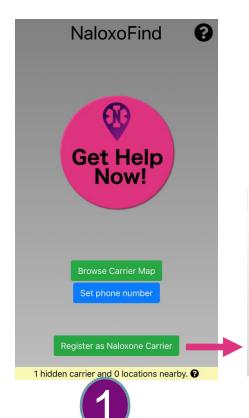

Altrix Medica

- Login with Facebook or
- Click "Register" to create a new account (email verification required).
- Click "Yes" to allow Naloxofind to provide alerts and access to your motion.

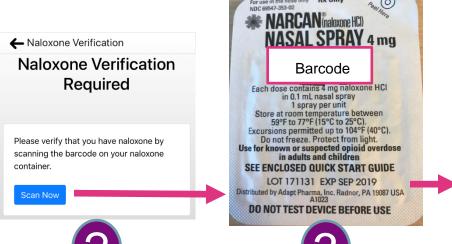








Naloxone Carrier – Registration (Cont'd.)

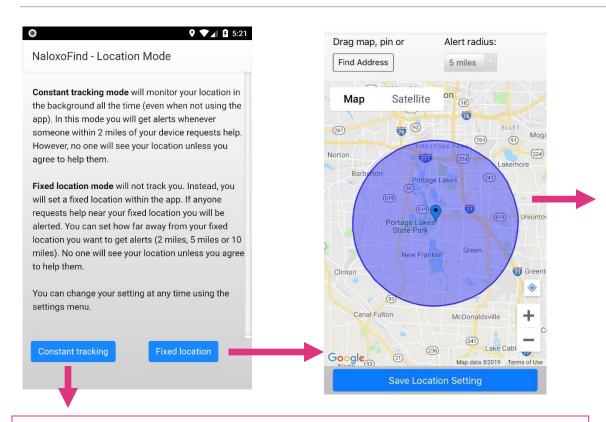


Constant tracking (recommended): NaloxoFind monitors your location and alerts you if someone within a two mile radius requests help. No one can see your location or contact you unless you accept a request.

- Fixed location: Drop a pin at a specific location to be alerted if help is requested within a radius you specify.
- Click "Find Address" and select an alert radius to drop the pin at a certain place (e.g., "Liberty Island").
- Drag the screen or pin to position by hand.
- Save Location Setting.
- No location will ever be sent from the device unless you accept a request for help.




Naloxone Carrier – App Navigation

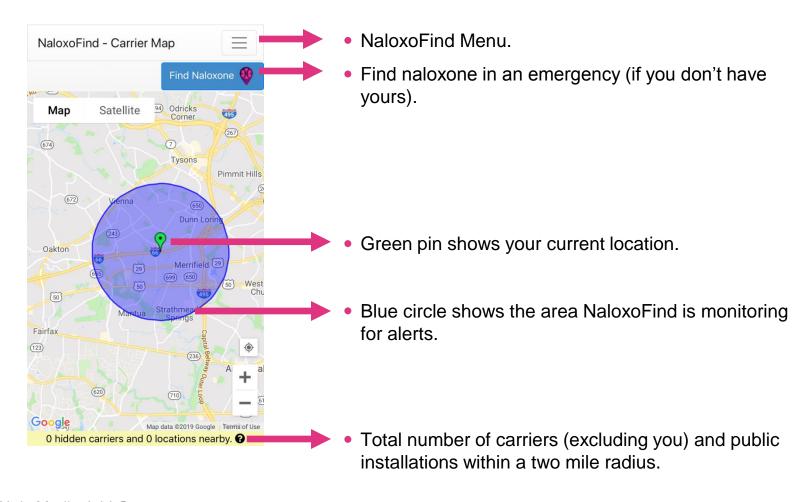




Naloxone Carrier – Fixed vs. Constant

Fixed Location Mode

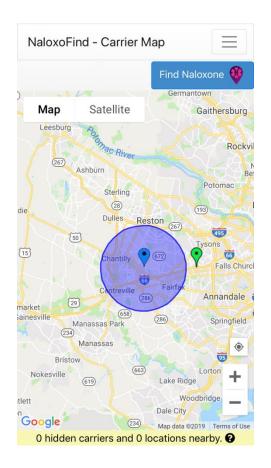

- Blue pin shows the fixed location that is being monitored.
- Blue circle shows selected alert radius.
- Green pin shows your current location.


Constant Tracking Mode

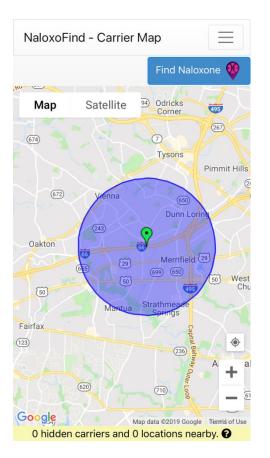

- Blue circle shows a two mile radius – alerts sent within this circle will be received by NaloxoFind.
- Green pin shows your current location.



Naloxone Carrier – NaloxoFind Menu

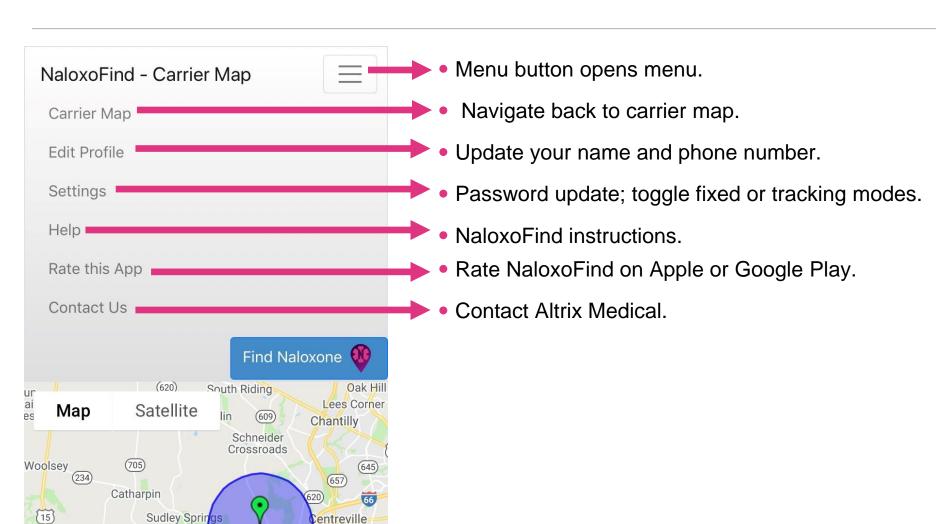



Sudley Sprin


entreville

Naloxone Carrier – Alerts



 Alert location. Clicking pin provides directions to the location.

Accept or decline request.
 On acceptance, your location and phone number are shared with requestor and other responders.

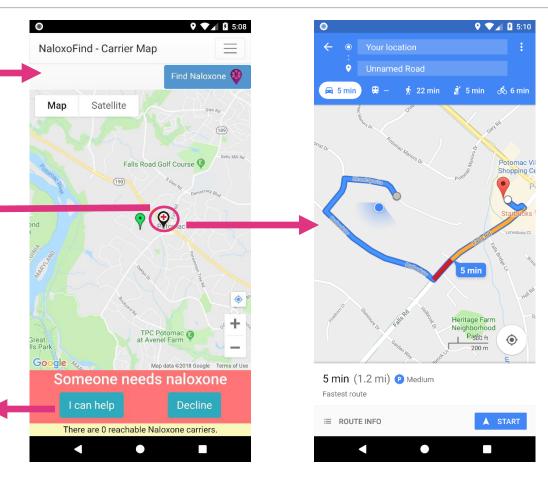



Naloxone Carrier – Accepted Alert

♥ ▼ 🖈 🛭 5:10 NaloxoFind - Carrier Map Stop helping. Helping in progress Stop Helping Road Golf Course 🚱 Satellite Map Registered public location. You are here. Potomac Person requesting naloxone. Click for directions. Google Map data @2018 Google Terms of Use Remember to put on your gloves before NaloxoFind helping Call this person. **©**17:09 Anonymous 2 need help! **Q**17:10 Mary Doe Coming to help! Type your message here... Group text messaging between There are 0 reachable Naloxone carriers. requestor and all responders.

Questions, Comments, and Feedback

naloxofind.feedback@altrixmedical.com



